


SciCan STATIM MAINTENANCE TIPS

STATIM CLEANING

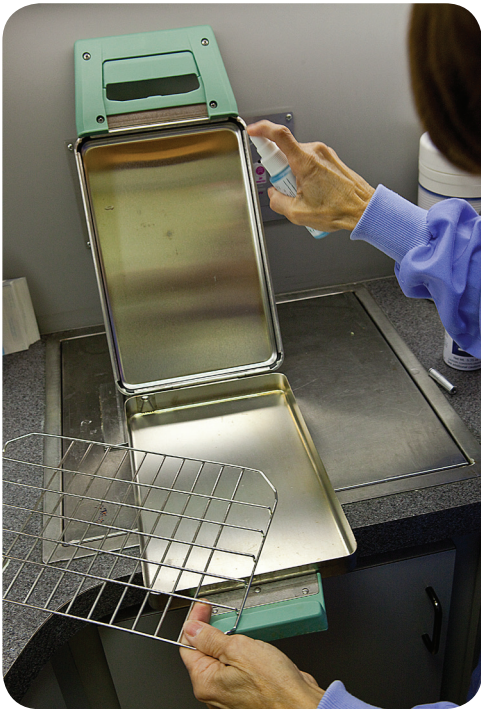

You should **CLEAN THE INTERIOR SURFACE OF YOUR STATIM CASSETTE** at least once a week with a mild, chlorine-free dishwashing soap. To promote quick-drying, spray the cassette lid with **STAT-DRI PLUS** after each cleaning.

PROPER PLACEMENT OF AUTOCLAVE BAGS



If you are sterilizing your handpieces in half paper/ half plastic autoclave bags, remember to **PUT THE PAPER SIDE UP FOR CHAMBER-TYPE STERILIZERS AND THE PAPER SIDE DOWN FOR STATIM CASSETTE STERILIZERS. THIS RESULTS IN EASY EVAPORATION** and longer handpiece life.



STAT-DRI PLUS DRYING AGENT

Stat-Dri Plus protects the interior of your Statim cassette and **SHORTENS YOUR DRYING TIME.**

- Stat-Dri Plus 32 oz. Refill with Rinse Aid S01-32OZPLUS
- Stat-Dri Plus 8 oz. Sprayer Bottle with Rinse Aid ...S01-8OZPLUST
- Stat-Dri Plus 8 oz. with Rinse Aid..... S01-8OZPLUS
- Stat-Dri Plus 2 oz. Sprayer Bottle S01-2OZPLUS
- Stat-Dri Plus Plates (Package of 5)S01-103935

HOW TO DRY YOUR STATIM IN 15 MINUTES



1 Bubble at 5 o'clock

2 Spray Stat-Dri Plus

3 Paper side down



4 Make sure the exhaust tube is free from kinks and following gravity

5 No overloading

HAYES

REPAIRS | SALES | TRAINING

Contact your local Hayes office for more Statim maintenance & repair information!